



Reference Guide

No Coding Required
 Sample Size – 0.5 µL
 Test Time – 6 sec.*
 Memory – 1865 Test Results

IMPORTANT: See your BGStar® Owner's Guide for a more detailed step-by-step guide on how to perform the test.

With your BGStar® System, you can test your blood glucose on your fingertips, palms (at the base of the thumb) or forearms (see Owner's Guide for exact locations).

*Average test time.

About Reference the BGStar® System




Blood Glucose Meter

- 1) Display
- 2) CHECK Button
- 3) Backlight Button
- 4) DOWN Arrow Button
- 5) UP Arrow Button
- 6) Strip Port



Test Strip

- 1) Contact Bars
- 2) Visual Fill Window
- 3) Sample Area

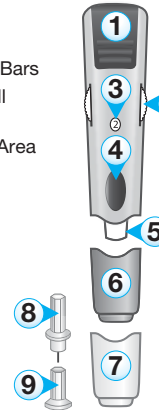

Lancing Device

- 1) Cocking Handle
- 2) Depth Adjustment Dial
- 3) Depth Indicator Window
- 4) Release Button
- 5) Lancing Holder Cup
- 6) Lancing Device Cap
- 7) Alternate Site Testing Cap
- 8) Lancing Device
- 9) Lancing Device Cover

Blood Glucose Testing Steps

(For complete instructions, see your BGStar® Owner's Guide)

1 Insert the test strip into the meter:

Insert the test strip, with the contact bars facing up, into the test strip port. The meter will automatically be activated.


CAUTION: To ensure accurate results, wash your hands with warm, soapy water and dry before every test.



2 Lance the test site:

Reference the instructions for how to prepare the lancing device, on the right.


Press the lancing device against the site to be lanced. Press the release button.

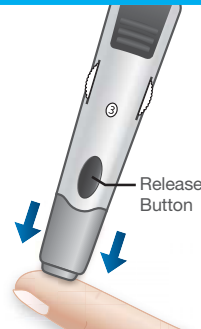

3 Obtain blood drop:


Wipe away the first blood drop that appears using a clean tissue or paper towel.

Squeeze from the base of your finger up towards the tip, until a second small blood drop forms.

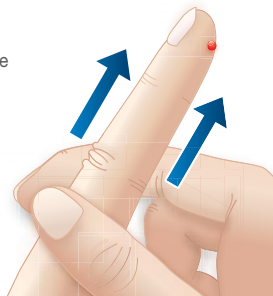


4 Bring test strip to blood sample:

The test strip draws the blood into the test strip through the sample area.

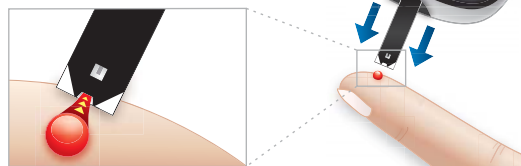

5 Remove Test Strip from Blood Sample when you hear the beep:

Your meter will beep once, when you should remove the tip of the test strip from the blood sample. The scrolling bar and 1-2-3 indicate that the meter is calculating your results.

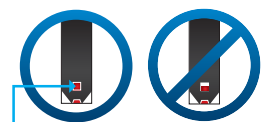


The entire visual fill window will turn red.



6 Final Steps:


Your test is completed when your blood glucose test result with time and date is shown on the display. Pressing the CHECK button confirms the mealtime tag and it stops flashing.

Remove the used test strip. The meter will automatically deactivate. Snap off the cap from the lancing device. Replace lancing device cover. Remove the used lancet. Discard the used test strip and lancet properly.

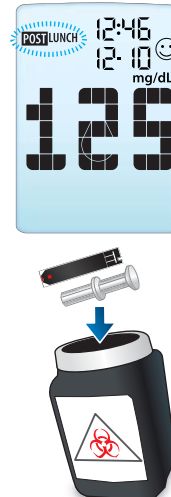


Preparing the Lancing Device


1 Snap off the cap from the lancing device.

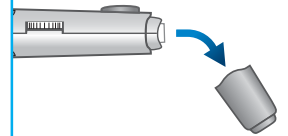


2 Insert a new lancet firmly into the lancing holder cup.

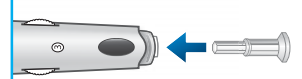


3 Twist the lancet cover off.

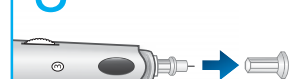

4 Replace lancing device cap.

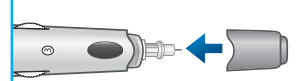

5 Set the lancing level.

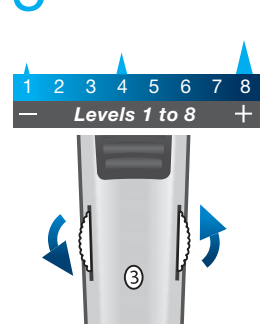

6 Pull the cocking handle out until it clicks.

